English

Disposable Webbed* EEG [Electroencephalography] Electrode

Intended Applications

EEG, EP [Evoked Potentials], IONM [Intraoperative Neurophysiological Monitoring], ICU [Intensive Care Unit], LTM [Long Term Monitoring], PSG [Polysomnography] and Ambulatory.

Sterilization

Non-sterile.

Caution

Disposable Webbed EEG Electrodes are for professional use only and should only be used in compliance with accepted industry standards. RhythmLink International, LLC is not responsible for injury, infection or other damage resulting from the use or misuse of this product.

Packaging

Ten Electrodes per pouch.

Instructions for Use

Clean application site. Apply Electrode using conductive gel, paste, collodion or other similar adhesive. When finished, remove Electrode and clean application site. This product is single-patient use only. Discard Electrode after use.

This product has a useful shelf life of 5 years from the date of manufacture.

RhythmLink International, LLC
1140 First Street South
Columbia, SC, USA 29209-3540
+1.866.633.3754 [toll-free]
+1.803.252.1222
+1.803.252.1111 [fax]
sales@rhythmink.com
Rhythmink.com

European Authorized Representative
Mdi Europa GmbH
Langenhagener Str.71
D-30855 Hannover-Langenhagen

**FOR SINGLE USE ONLY
ASSEMBLED IN CHINA**

Caution: Federal Law [USA] restricts this device for sale by or on the order of a licensed medical practitioner.

RhythmLink® is a registered trademark of RhythmLink International, LLC.

*US Patent No. D 644,738 and European Patent Registration No. 001893751-0001.

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RhythmLink® è un marchio registrato di RhythmLink International, LLC.
RhythmLink® ist eine eingetragene Marke von RhythmLink International, LLC.